

DE-500 DERMATOSCOPE

User Manual



The following directions for use will enable you to familiarize yourself with the device. Please review them carefully!


Intended Use

The device is an internally powered medical microscope for skin lesions diagnosis. It can be used for both contact and non-contact skin examination with its particularly lighting system.

The universal lens mount helps connect dermatoscope with smartphone or tablet to capture and display images.

Product Overview

- ① Back Contact Ring
- ② On/Off/Polarized/Non Polarized
- ③ UV Mode
- ④ Polarized Mode Indicator
- ⑤ Charging Connector
- ⑥ Contact Plate

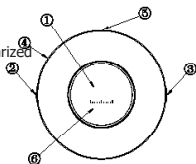

Scope of Delivery

The set consists of the dermatoscope with universal lens mount, metal dual lens mount, USB-C cable, carry case, clean cloth and a user manual.

Technical Data

Material	Aluminum & Glass
Optical Design	All glass, 4 elements 3 groups, anti-reflection coating
Lens Diameter	25mm(front); 15mm(rear);
Magnification	10X
Distortion	5%
Working Time	90 minutes
Polarization	16 LEDs, 8 polarization, 4 non polarization, 4 365nm UV light
Resolution	300 LP/MM (Axis) 280 LP/MM (Edge)
LED Type	SMD LED beads
Battery Capacity	300mAh Lithium ion



charge the instrument fully before first use.

Warning: Use a charger with a power output of 30W or less.

Operation

To switch on the illumination, press the button (②, ③) on the instrument. The device starts with white polarized mode.

● On/Off

Push on/off ② button one second to turn on the dermatoscope. Long press 3 seconds to turn off the dermatoscope

● **Polarized/Non Polarized**

The default mode is polarized light illumination when turn on, polarized mode indicator ④ turn on. Press again to switch to non polarized mode, polarized mode indicator turn off. Circulating.

● **UV Mode**

Press UV mode ③ to turn on 365nm UV light. This mode should be used after the device is powered on.

Warranty

The device is maintenance free. The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 2 years.

Disposal Instructions



The device may not be disposed of as domestic waste. Dispose of the device in a waste facility that accepts electronics/electrical appliances.